

# ANABOX<sup>®</sup> 7x4

The medication dispenser for one week with 28 individual cups-compatible with solid and liquid medicines.

- **safe**
- **dishwasher-safe**
- **overseeable**
- **airtight & waterproof**





## INSTRUCTIONS FOR USE

Class I Medical  
Device  
Version: 01/2022

### Taking out the medication

The day of intake (columns) and the times of intake (rows) are printed on the tray.

1. Take out the cup of the corresponding time of intake (“morning”, “noon”, “evening” or “night”) and the corresponding day of intake (Monday-Sunday).
2. In order to open the cups, simply push the protrusion of the cover upwards.
3. For a simplified intake of the medication directly from the cup, use the mouthpiece provided for this purpose.
4. After the medication intake, put the cup back into the opening of the tray.

### Transport

For a safe transport, use the cover included in the scope of delivery, simply pushing it onto the tray.

Alternatively, it can also be used as a bowl by inserting the tray into it.

### Cleaning

1. Remove the cup and open the cover.
2. Rinse the cup with lukewarm water and a mild cleaning agent appropriate for cleaning dishes.
3. Alternatively, you can put the tray including the cups with the covers open into the dishwasher (operating temperature 50 °C).
4. After both cleaning methods, make sure that the cups have been sufficiently dried before refilling with medication.

### Filling

1. Insert all empty, cleaned and dried cups into the openings of the tray and open the cups starting from the top (row “morning”).
2. Now fill each cup with the appropriate medication according to the day and time of intake. As soon as the medication is in the cups, close the covers by pressing lightly.

### General

- Keep the medication dispenser out of the reach of children!
- Clean the cups before first use or refilling according to the instructions.
- Comply with any specific storage conditions for medicines such as e.g. light and temperature sensitivity, etc. according to the package insert in the respective packaging. In case of doubt, please ask your pharmacist or the medicine manufacturer.
- We accept no liability for the incorrect use or the incorrect intake of medication.
- The use by people with mental disabilities shall take place only under expert supervision.
- Based on the production date, the product should be used for a minimum of 5 years if used properly, if stored and cleaned as instructed.